

DuoSafe Sideguards


De DuoSafe sideguards provide reliable fall protection for the patient. The integrated touch panels make it possible for both the patient and nursing staff to adjust the bed functions.

Construction and usability

- The sideguards consist of 4 synthetic panels that have been installed on swivel arms.
- The steel swivel arms guarantee an optimal strength.
- The sideguards are installed with bearings, so that a limited sound volume and maintenance is guaranteed.
- The sideguards are installed on the backrest and lying frame, and offer optimal patient protection, even when the mattress platform sections have been inclined.
- The panels have no sharp corners.
- Ergonomically shaped handles provide a pleasant grip.
- Each panel can be raised separately. It is automatically locked in its highest position.
- A damping ensures that the panels slowly come down.
- In their lowest position, the sideguards are no obstacle when the patient wants to get out of the bed or has to be transferred. This also is the case when patient lifts have to be used or when the bed has to be made up.
- The gaps between the panels meet the strictest standards and ensure a high level of patient protection.
- Measured from the mattress platform, the total height is ca. 42 cm.
- The distance from the top of the sideguards in their lowest position to the mattress platform is max. 7,5 cm.
- The sideguards are equipped with an integrated accessory hook on both bed sides.
- The sideguards also serve as an raising aid.
- The brake pedal of the ped is accessible at all times, even when the sideguards are down, thanks to a 32 cm opening between the foot panel and the sideguard at the foot end.
- The DuoSafe panels have two integrated clinometers that indicate the inclination of the backrest and mattress platform.

Built-in keypads (*)

- **Keypad - nursing (*)**: the nursing keypads are located on the outside of the DuoSafe sideguards and are equipped with all necessary functions to bring the patient into the desired sitting, lying or reading position. Additional functions allow the nursing staff to lock functions and to check the safe bed position and battery status in a blink of an eye.
- **Keypad - patient (*)**: The patient keypads with an integrated backlight make it possible to control bed functions via an easy accessible touch panel.

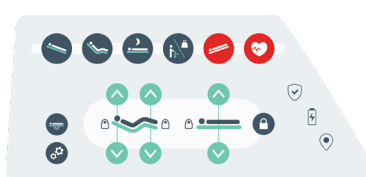

Colours decoration strip

Misty mint
Stone grey


(*) Option



Dimensions

DuoSafe sideguards

- Panel 1: 94 x 42 cm
- Panel 2: 71 x 42 cm
- maximum height of the mattress: 19 cm

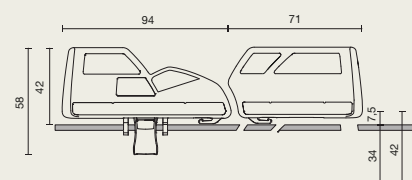

Materials and finishing

- Synthetic material.
- Resistant to the common cleaning products.

The DuoSafe sideguards comply with the **norm EN 60601-2-52**.

For further information about **used materials, constructions and maintenance**: consult our materialfiles.

Options

- Built-in keypads to activate bed functions.

